Vagus™ Manual

Instructions

Customer instructions

Thank you for purchasing Vagus™ for identification of autonomic neuropathy. Please read this manual carefully before using Vagus™ for the first time. Failure to comply with these instructions may lead to incorrect measurements and may cause damage to Vagus™. We take no responsibility for any personal injury in connection with the use of Vagus™.

Vagus™ Patient instructions

- Do not use Vagus™ if the patient has a pacemaker.
- Using the device will not harm the patient or damage the pacemaker, but the pacemakers cardiac pacing will interfere with the value of the Vagus™ algorithm.
- If the patient is undergoing laser treatments for proliferative retinopathy step 4, VM/Vasalva, should be avoided due to a hypothetical risk of bleeding in retina.
- The patient's heartbeat must be in a state of sinus rhythm.
- The patient must not smoke or drink any liquids containing caffeine before the measurements.
- It is recommended that the patient fasts two hours prior to the measurements. The patient should wash his/her hands with water and soap before the examination. Do not use alcohol-based hand rub since this will dry out the patient's hands. You may moisturise the patient's hands with water.
- In order to take care of the measurements as well as Vagus™, the patient must not wear rings or have any wounds on his/her hands.
- The patient must not speak or make sudden, non-instructed movements during the measurements.

Field of application (Intended use)

Vagus™ is a medical device for screening of autonomic neuropathy in adults (>18 years). Vagus™ must be used by a physician or instructed staff. Vagus™ must not be used by patients for self-diagnosis.

Warnings/Safety instructions

- Be aware that some medication may affect the autonomic nerve system.
- Use only authorised components for this device.
- The mouthpiece must be used only for one patient examination.
- Do not use this device on pacemaker patients.
- Do not use this device and a defibrillator at the same time.
- Do not use this device near flammable anaesthetics, drugs or pressurised oxygen (e.g. a pressure chamber or ultraviolet steriliser).
- Do not use this device on patients with sensitive skin and/or allergies due to risk of skin irritation or rash.
- Keep out of the reach of children and persons who cannot express their intentions.
- Do not use this device for any other purpose than screening of autonomic dysfunction.
- Do not expose this device to severe shock or vibration.
- Do not use this device near strong electromagnetic fields.
- Do not use other types of batteries than stated in this manual.
- Remove the batteries if Vagus™ is not going to be used for a longer period, e.g. for one month.

Caution

- Avoid transmission of static electricity before use.
- Always install the batteries according to the instructions on the battery cover.
- The battery cover and the end cover **MUST** be replaced before use.
- Use only ALKALINE batteries (LR6).
- Make sure that at least 1/3 of the battery life is shown on the battery indicator.
- Check on the display screen that there is sufficient storage on the SD memory card, e.g. 480/122, (storage for 122 measurements).

Vagus™ Test kit

In general

- Usage of this device is recommended at a temperature between 18°C and 25°C and with a relative humidity below 80%.
- Do not use a mobile phone near this device.
- Do not sterilize this device.
- Do not disassemble, attempt to repair or modify this device.
- This device must not be exposed to or washed in water, acetone, ether, freon or petroleum derivatives.
- Always store Vagus™ in the accompanying storage case.
- Do not expose Vagus™ to direct sunlight, high air humidity or temperatures or to dusty, wet or moisturized surroundings. Do not store the device close to open fire, strong vibrations or electromagnetic fields.
- Dispose Vagus™ and its components according to applicable local regulations.

Test kit	Number	Qty	Lifespan / Capacity	Replacability / Ordering	Disposal
Vagus™ device	V01.1	1	5 years	Can be reordered	Recycling centre
Elektrodes	V02.1	2	50 measurements	Replacable/Can be reordered	Ordinary waste disposal
Valsalva mouthpiece	V03.1	2	Single-use only	Replacable/Can be reordered	Ordinary waste disposal
Manual	V04.1	1	-	Can be reordered	Ordinary waste disposal
Test pad	V05.1	1	-	Can be reordered	Ordinary waste disposal
SD memory card	V06.1	1	480 measurements	Replacable/Can be reordered	Recycling centre
AA (LR6) batteries	V07.1	2	500 meas. per year	Replacable/Can be reordered	Recycling centre
Storage case	V08.1	1	-	-	Ordinary waste disposal

Table of contents	
Instructions	2-3
<hr/>	
Instructions, Warnings and Vagus™ Test kit	
Vagus™ Introduction	4-5
<hr/>	
Early detection of autonomic neuropathy	
Usage and navigation	
Using Vagus™ for the first time	
Before the examination	
How to handle Vagus™	
Vagus™ Test procedure	6-8
<hr/>	
Step 1 . HR, Step 2 . RS, Step 3 . E:I and Step 4 . VM	
Shutdown, cleaning and storage	
Vagus™ Device symbols	8
<hr/>	
Information and Sources of error	9
<hr/>	
Vagus™ Service / Replacements	10-11
<hr/>	
Replacement of electrodes	
Replacement of batteries	
Clearing the SD memory card	
Note	
Vagus™ Contact	12
<hr/>	
Vagus™ Specifications	12
<hr/>	

Vagus™ Introduction

Early detection of autonomic neuropathy

Vagus™ enables an easy and early detection of autonomic neuropathy using a simple 4-step test, four heart rate measurements, which altogether takes less than 15 minutes. If one of the four steps produces an abnormal result, this indicates autonomic dysfunction (early stage) – if two or more steps produce abnormal results, the diagnosis autonomic neuropathy can be made.


Neuropathy is a common complication in diabetes, though not only in diabetes. The symptoms of neuropathy may be weak and uncharacteristic for years and are therefore easily overlooked. Thus, the disease may be diagnosed very late and unfortunately often not until permanent nervous damage is recognized. Internationally, autonomic neuropathy screening is recommended at the time of diagnosis in patients with type 2 diabetes and five years after the time of diagnosis in patients with type 1 diabetes. Hereafter, yearly screenings of both groups are recommended.

The underlying technology of Vagus™ is a standard 1-lead ECG system recording the heart rate during the 4-step test. Vagus™ is a hand-held device with a display that guides the patient and the examiner through the 4-step test.

Regardless of other complications, the presence of autonomic neuropathy is a bad prognostic symptom and should change the stratification level for the patient and lead to decisions on diagnosis, treatment and future control.

Usage and navigation

Vagus™ applies the hands as "electrode positioning" instead of ordinary disposable ECG electrodes. Vagus™ is turned on by pressing  for two seconds. Start the test by pressing  and follow the instructions on the display. Choose step of the test by pressing  or . Disconnect or return to previous step by pressing . Vagus™ is turned off by pressing  for two seconds. Vagus™ will automatically turn off after two minutes of inactivity.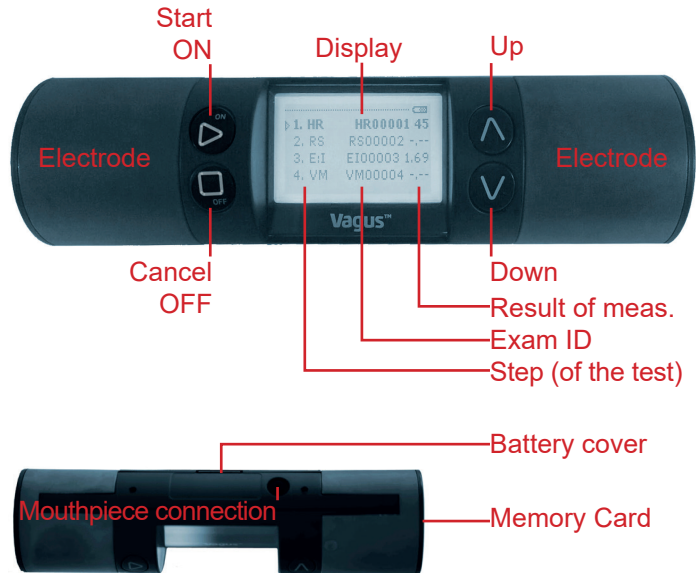

Using Vagus™ for the first time

Make sure that all objects are placed in the Vagus™ test kit, cf. the table, page 3. Install the batteries, cf. page 8. Turn on Vagus™ by pressing  and make sure that the Vagus™ logo appears on the display. Select language. Vagus™ is now ready for use. You can also select language(, e.g. Danish or English,) under 'Settings' on the display. Exam ID is automatically assigned consecutive numbering.

Before the examination

The patient should wash his/her hands with water and soap before the test begins. The patient must not wear rings during the measurements. The patient must not speak or make sudden, non-instructed movements during the measurements. The patient must not smoke or drink any liquids containing caffeine before the measurements. It is recommended that the patient fasts two hours prior to the measurements. Check the display to ensure that at least 1/3 of the battery life is shown on the battery indicator.

How to handle Vagus™

Do not squeeze Vagus™ during the test. Both hand position A and B is applicable for the test.

Vagus™ Test procedure

Step 1 . HR

5 min.

Measuring heart rate while resting in a supine position.

- The patient rests for five minutes without speaking or moving.
- Choose 1. HR on Vagus™.
- Place Vagus™ in the palms of the patient.
- Press  and Vagus™ counts down and then starts the measurement.
- The measurement lasts for five minutes, short beep = approved.
- The display shows an Exam ID, a consecutive number starting with HR (e.g. HR34387), and the result of the measurement (e.g. 44).
- Note down the Exam ID and the result of the measurement on the test pad.


A long beep (two seconds) = Error measurement.

See sources of errors on page 9.

This generally applies for all four steps. In case of error measurement, no result is obtained.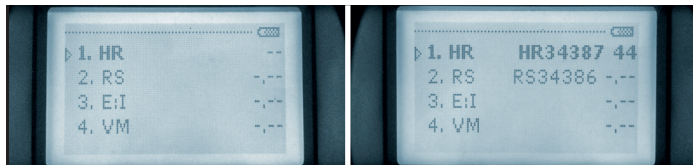

Repeat the step, no more than three times though, and then continue to the next step.


Step 2 . RS

1 min.

Measuring the heart rate response from supine to upright position.

- The patient must lie still after the HR measurement.
- Choose 2. RS on Vagus™.
- Place Vagus™ in the palms of the patient.
- Press  and Vagus™ counts down.
- When the device "beeps" the patient must stand up holding Vagus™ in his/her hands.
- If the patient is not able to stand up with Vagus™ in his/her hands, Vagus™ must be handed over when the patient is in an upright position.
- Press  again as soon as the patient is standing upright with his/her arms hanging relaxed.
- The measurement lasts for one minute, short beep = approved.
- The display shows an Exam ID, a consecutive number starting with RS (e.g. RS34388), and the result of the measurement.
- Note down the Exam ID and the result of the measurement on the test pad.

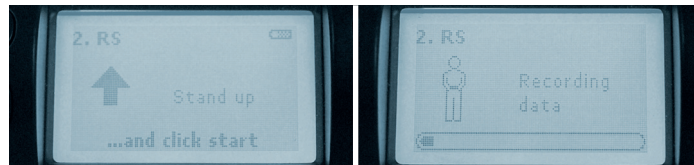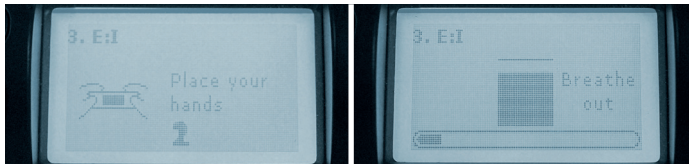
Step 3 . E:I

1 min.

Measuring the heart rate response during deep breathing.

- The patient must be sitting in a relaxed manner.
- The patient must follow the instructions on the display while slowly breathing in and out.
- Choose 3. E:I on Vagus™. Place the device in the hands of the patient.
- Press  and Vagus™ counts down and starts the measurement.
- The examination starts with the patient inhaling through his/her nose until inhaling is no longer possible. When the text changes and the bar moves downwards, the patient must passively exhale through his/her mouth following the pace of the bar.
- The measurement lasts for one minute, short beep = approved.
- The display shows an Exam ID, a consecutive number starting with EI (e.g. EI34389), and the result of the measurement.
- Note down the Exam ID and the result of the measurement on the test pad.

Step 4 . VM

1 min.

Measuring the heart rate response during expiration with a regular pressure of 40 mmHg and during rest.

- The patient must be sitting in a relaxed manner.
- Insert the tube from the Valsalva mouthpiece to the connection of the Vagus™ manometer.
- Choose 4. VM on Vagus™.
- Place the Valsalva mouthpiece in the patient's mouth and Vagus™ in his/her hands.
- Press  and Vagus™ counts down and starts the measurement.
- Follow the instructions on the display – the patient takes a deep breath, then forcibly exhales for 15 seconds through the mouthpiece.
- Remove the mouthpiece still holding Vagus™ in the hands.
- The measurement lasts for one minute, short beep = approved.
- Note down the Exam ID (e.g. VM34390) and the result of the measurement on the test pad.

If the patient is undergoing laser treatments for proliferative retinopathy, step 4 should be avoided due to a hypothetical risk of bleeding in retina.

Vagus™ Test procedure (continued)

Shutdown, cleaning and storage

Turn off Vagus™ by pressing . After each examination, carefully clean Vagus™ once with a disposable wet alcohol or disinfection wipe, such as "Wet Wipe PHMB Disinfection". Then use a dry, soft cloth and make sure that the electrodes are completely dry before the device is placed in the storage case. Always keep Vagus™ in the storage case when not in use. Always remove all batteries if Vagus™ will not be used for one month or longer.

Warning

Do not clean Vagus™ using acetone, ether, freon or petroleum derivatives. Do not submerge Vagus™ in any fluids. Do not subject Vagus™ to extreme temperatures, humidity, dust, strong shocks or vibrations.

The batteries must not be in direct contact with soap or water.

Vagus™ Device symbols

This product is a Class BF electrical device, i.e. it meets the required level of protection against leakage current and it has an isolated (floating) connection to the patient.

Manufacturer.

Important safety instructions. Read the manual before operating this device.

Read the manual before operating this device.

Vagus™ contains electronic parts and must be disposed of according to applicable local regulations.

CE marked by the manufacturer and approved by an authorized agency.

Caution.

Warning.

Vagus™ Information and Sources of error

Display / Sound	Cause	Solutions / Recommendations
Vagus™ (no sound)	Vagus™ is turned on and ready for use.	1. Examination can be started.
Vagus™ cannot be turned on	Worn-out batteries.	1. Replace with new AA (LR6) batteries. 2. Check that the batteries are installed properly. Refer to 'Replacement of batteries', page 10. 3. Check that the SD memory card is installed properly.
The SD memory card is full (no sound)	Full memory card.	1. Clear the SD memory card. Refer to 'Clearing the SD memory card', page 11.
Warning. Unstable beats (Short beep) 	Possible cause: Abnormal heart rate.	1. Repeat the measurement. However, no more than three times. 2. Conduct a standard ECG examination.
Error. Data error (Long beep) 	Possible cause: Signal interference. The data quality is insufficient.	1. Hold your arms and hands still. You may moisten your hands. 2. Repeat the measurement. However, no more than three times. 3. Replace the electrodes. Refer to 'Replacement of electrodes', page 10.
Error. Low pressure (Long beep) 	Possible cause: The patient exhales below the predetermined value of the VM test.	1. Repeat the measurement. However, no more than three times.
Error. Result error Resultatfejl (Long beep) 	Possible cause: The result is outside the measurable area.	1. Repeat the measurement. However, no more than three times. 2. Conduct a standard ECG examination.

Vagus™ Service/Replacement

Replacement of electrodes

Electrodes will gradually become lighter and lose the conductivity ability, and it is recommended that the electrodes are replaced after approximately 50 patient examinations or when they become light or heterogeneous on the surface.


Turn off Vagus™ by pressing  for two seconds.
Follow these instructions:

1. Unscrew the left end cover.
2. Carefully remove the worn-out left electrode.
3. Carefully place the new electrode labelled with “left”.
4. Screw the end cover back on.
5. Repeat the procedure on the right side.
6. Dispose of worn-out electrodes using ordinary waste disposal.

Replacement of batteries

The lifespan of the batteries should be enough for app. 500 measurements. Replace the batteries if the battery symbol indicates low battery or if a prompt is displayed. Turn off Vagus™ and turn the device until the battery cover is in front of you. Follow these instructions:

1. Open the battery cover by pulling the latch in the middle.
2. Note the battery type and +/- positions. Remove the worn-out batteries.
3. Install the new AA (LR6) batteries so the + (positive) and - (negative) polarities match the polarities of the battery compartment as indicated.
4. Replace the battery cover.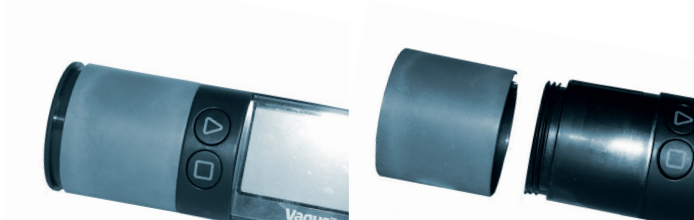

Clearing the SD memory card

Vagus™ memory card is sufficient for 480 measurements, and must then be cleared. Turn off Vagus™ by pressing  for two seconds. Follow these instructions:

1. Remove the batteries and unscrew the right end cover.
2. Push in the SD memory card to release it and then pull it out.
3. Insert the SD memory card into a Mac or PC. The card will be cleared while data are being transferred or deleted. Do not remove the file: `_SETUP_.text` from the SD memory card. Insert the memory card back into the slot on Vagus™. There is only one way to insert the card.
4. Screw the end cover back on.
5. Install the batteries and replace the battery cover.
6. Turn on Vagus™.

You need an SD card reader and a Mac or PC for clearing the SD memory card.

Note

Replacement of batteries, SD memory card and electrodes may be carried out by a physician or instructed staff. Then, make sure that Vagus™ is working properly by turning on the device. Press  so that the Vagus™ logo appears on the display.


Vagus™ does not work without the SD memory card. If the device is turned on without the SD memory card inserted, the display will light up, but the Vagus™ logo will not appear. Follow these instructions:

1. Remove the batteries and unscrew the right end cover.
2. Reinsert the SD memory card. There is only one way to insert the card.
3. Screw the end cover back on.
4. Install the batteries and replace the battery cover.
5. Turn on Vagus™.

If, by mistake, the file: `_SETUP_.text` has been deleted from the SD memory card, "Exam ID" and all settings will be reset. Repeat the procedure "Using Vagus™ for the first time", refer to page 5.

Contact support@medicusengineering.com for help.

Do not disassemble or attempt to repair Vagus™.

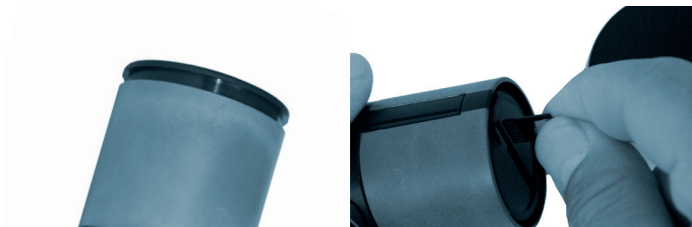Vagus™ Contact

Order placemets

Please place your order on our website: vagusdevice.com
(Item Number V03.1) Valsalva mouthpieces
(Item Number V02.1) Electrodes
(Item Number V05.1) Test pads

Or place an order to: order@medicuseng.com

Provide name, address, phone number, EAN-number and item number and specify the number of pads, mouthpieces and electrodes requested.

Support

If you need guidance or external service please contact Medicus Engineering ApS for technical support:
support@medicuseng.com

Other enquiries

For other enquiries please write to:
mail@medicuseng.com

Address

Medicus Engineering ApS
Palle Juul-Jensens Bvd 82 . DK-8200 Aarhus N
vagusdevice.com
medicuseng.com

*Do not copy or reproduce any parts of this manual without prior written permission from Medicus Engineering ApS.
Copyright © 2019 Medicus Engineering ApS.*

Vagus™ Specifications

Product name:	Vagus™
Medical equipment:	Ila
Product dimensions:	L: 169 mm / Ø: 45 mm
Product weight:	120 g not including batteries
Display:	LCD display with backlight
Batteries:	2 AA batteries, (LR6) 1.5V
Electrodes:	Silver/silver chloride
Measurement method:	1-lead ECG measurements
Measurement range, pressure:	0 to 66 mmHg
Measurement range, pulse:	28 to 200/min.
Memory card capacity:	480 measurements
Pressure accuracy:	±3 mmHg
Pulse accuracy:	±1% of reading
RR sampling frequency:	1000 Hz
Operating humidity:	<80% RH
Operating temperature:	5°C to 40°C
Atmospheric pressure:	80 to 106 kPa
Storage humidity:	<80% RH
Storage temperature:	-20°C to +50°C
Transportation humidity:	<80% RH
Transportation temperature:	-20°C to +50°C

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